



VELA Salsa

VARIANT:

100/110/120/130/400

Manual no. 105032



VELA Salsa 100



VELA Salsa 110



VELA Salsa 130



VELA Salsa 120



VELA Salsa 400

TABLE OF CONTENTS

1.0. INTRODUCTION	3
1.1. SAFETY	4
1.1.2. Transport	4
1.2. WARRANTY	4
1.3. UNPACKING THE CHAIR	5
1.4. MAINTENANCE	5
1.4.1. Wheels	5
1.4.2. Frame	5
1.4.3. Upholstery	5
1.5. USER MANUAL	6
1.5.1. Adjustment of seat height	6
1.5.2. Adjustment of seat height (Salsa 130 & 400)	6
1.5.3. Adjustment of seat tilt	6
1.5.4. Adjustment of seat tilt (Salsa 130 & 400)	6
1.6.5. Brake	7
1.5.6. Adjustment of back height	7
1.5.7. Adjustment of back tilt	7
1.5.8. Adjustment of armrest, height	7
1.6.9. Adjustment of armrest, width	8
1.6.10. Rotation (optional)	8

1.0. INTRODUCTION

DEAR CUSTOMER

Thank you for purchasing a VELA Salsa. This user manual contain useful information about positioning, operating and maintenance of VELA Salsa stools.

Important

Please read this user manual thoroughly and keep it for later use. This user manual must always accompany the chair. The user manual can also be found on VELA's website: www.vela.eu, where you also can find other relevant information about VELA Salsa stools.

If you have any questions, please contact VELA's distributor in your country.

VELA reserves the right to make changes to this manual.

VELA

Gøteborgvej 8-12
DK-9200 Aalborg
Denmark
www.vela.eu

GENERAL

1.1. SAFETY

This chair bears the CE mark and conforms to all applicable EU rules. The stool is manufactured by VELA, Vermund Larsen A/S, Goeteborgvej 8-12, 9200 Aalborg SV, Denmark.

VELA Salsa is an indoor stool designed for sitting persons.

VELA Salsa 100/110/120/130/400 is tested and approved for a maximum user weight of 125 kg.

1.1.1. TRANSPORT

By transport in car, by plain or other means of transport, always use the chair's brake. The chair is not approved as a passenger seat in means of transportation and should not be used as such.

The chair withstands the following transport specifications:

Temperature: -40°C (-40°F) to

+70°C (+158°F)

Humidity: 15-95%

Air pressure: PPP up to 1000 meters (3280 ft) above sea level

1.2. WARRANTY

VELA provides warranty by the laws and regulations which are debt in the country the chair is bought in. The warranty only applies when using original spare parts and accessories. Modification may only be carried out by authorised service technicians.

VELA does not accept responsibility for damage or injury to the product or user, which have arisen due to:

- :: Transportation
- :: Misuse
- :: General wear and tear
- :: Incorrect use
- :: Use of non-original spare parts and accessories
- :: Adjustments carried out by unauthorised service technicians

1.3. UNPACKING THE CHAIR

VELA Salsa is supplied in cardboard packaging, developed with consideration for the environment.

If there are any parts missing or transport damage to the chair, please contact your dealer immediately. Do not try to repair any damage or use the chair.

Content of the cardboard packaging:

- :: VELA Salsa chair
- :: This user manual

1.4. MAINTENANCE

We recommend your VELA Salsa undergoes an annual service to find, for example, defective or worn parts.

1.4.1. WHEELS

VELA recommend that the wheels are checked in regular intervals e.g. to remove possible thread remnants.

1.4.2. FRAME

Clean with water, possibly with a PH neutral soap.

1.4.3. UPHOLSTERY

Fabric: Clean with upholstery cleaners or wipe it with a tightly wrung cloth rinsed in clean water.

Imitation leather: Clean with water or mild soap.

Medical textiles: Clean as described above. Can also be cleaned with disinfectants and alcohol.

Leather: Clean by vacuuming it using a soft dusting brush. Avoid cleaning products.

1.5. USER MANUAL

Replacement of the gas spring may only be performed by the service support team from your dealer or a skilled technical person.



1.5.1. ADJUSTMENT OF SEAT HEIGHT

Adjust the seat height by pushing the front lever placed at your right hand.



1.5.2. ADJUSTMENT OF SEAT HEIGHT (SALSALSA 130 & 400)

Adjust the seat height by pushing the lever placed at your left hand.



1.5.3. ADJUSTMENT OF SEAT TILT

Adjust the seat tilt by pulling the middle lever placed at your right hand.



1.5.4. ADJUSTMENT OF SEAT TILT (SALSALSA 130 & 400)

Adjust the seat tilt by pushing the middle lever placed at your right hand.



1.5.5. BRAKE

VELA Salsa brakes by moving the lever on the side of the chair forward. Unlock the chair by moving the handle back into upright position.

Note: The brake can be moved by loosening the screw at the bottom of the brake lever, remove the plastic knob on the opposite side hole and insert the brake lever. Remember to tighten the handle again.



1.5.6. ADJUSTMENT OF BACK HEIGHT

Adjust the back height by using the lever placed on the back of the chair. Remember to tighten the handle again afterwards.



1.5.7. ADJUSTMENT OF BACK TILT

Adjust the back tilt by pushing the rear lever placed at your right hand.



1.5.8. ADJUSTMENT OF ARMREST, HEIGHT

Adjust by loosening the screw on the armrest bar and regulate the height of the armrest. Remember to tighten the screw again.



1.5.9. ADJUSTMENT OF ARMREST, WIDTH

Adjust by loosening the screw underneath the seat and regulate the width of the armrest. Remember to tighten the screw again.



1.5.10. ROTATION (OPTIONAL)

The seat is rotated by pulling up the forward handle on the left side.