culla-care by elmi

< therapy and relax

indoor

outdoor

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We would like to thank you for your trust and for the purchase of our product. **culla-care** is outstanding for its quality, its safety, its simple use and its modern design. In order to use this item properly, please read these user's instructions very carefully.

**Dear Customer,
please read this user's manual carefully before using culla-care for the first time!**

INDEX

Safety instructions	23
Warranty and product liability	24
Intended use	24
Components	25
Adjustment and operation	26
Assembly	27
Mounting of accessories	28
Types	29
Recommendations for cleaning	29
Technical data	30
Maintenance	31
Spare parts	31

SAFETY INSTRUCTIONS

culla-care is provided with the CE-marking and conforms to the requirements as laid down by the Council Directive 93/42/EEC

Norm UNI EN 12182 · Norm UNI En 1985 · Norm UNI EN ISO 14971

It has been developed following the latest regulations. The manufacturing process emphasizes continuous quality control.

Weight limit: 150 kg max user weight

Precautions!

culla-care should only be used if a qualified person or a family member is present.

culla-care should only be used by people who regarding their height and body constitution are able to assume a lying and seated position.

culla-care is suitable for indoor and outdoor use.

Please remove any objects from furniture that can be knocked over or fall down while using **culla-care**. The version **culla-care** indoor is suitable for indoor use and for outdoor use only on completely even ground. The version **culla-care** outdoor is suitable for indoor use and also for outdoor use due to its big rear wheels and its safety brake.

culla-care can only be used by people who are familiar with the instructions and applications of this user manual.

Please check that in the user's area of **culla-care** there are no obstacles such as stairs, steps or items of any kind lying on the floor that could cause **culla-care** to overturn.

Do not carry out any movements or interference, if you are not absolutely sure of their effects and consequences; in case of the slightest doubt please contact the nearest service centre or the manufacturer directly.

The application of the use of **culla-care** must be carried out with the support of qualified personnel. Pay attention to possible points of danger!

The use of **culla-care** and its accessories should be done under medical supervision.

Do not use **culla-care** on uneven ground, on slopes or on stairs.

The use of **culla-care** by unauthorized people – even if it is only for playing – is forbidden.

When stopping always block the brakes, even on level surfaces.

Before every use, check that the brakes function correctly; check that the casters have not become worn, and that all the accessories and components that could get loose are adjusted and properly tightened.

Do not systemize or leave **culla-care** on steep or uneven ground.

Do not use **culla-care** if parts are faulty or missing. Before use check that all the fasteners of **culla-care** are tightened.

culla-care should be cleaned as described in the „cleaning“ chapter.

In case of an accident or defects, contact your dealer who delivered **culla-care**.

Please keep these user's instructions for further consultation.

culla-care has to undergo service annually to assure long-term use by avoiding damage to the product and the individual. An authorized person should carry out the servicing.

WARRANTY AND PRODUCT LIABILITY

Elmi GmbH/srl grants a two years' warranty on **culla-care**.

Only original spare parts and accessories can be used. The guarantee becomes void if repairs or changes to **culla-care** or on its components are undertaken by unauthorized persons.

Every product responsibility of the manufacturer (Elmi Srl, Zona Artigianale 25, I-39052 Caldaro) is in this case excluded beforehand.

The manufacturer is not responsible for damages to persons, animals or things in the following cases:

- improper use of **culla-care**
- use of **culla-care** by unauthorized persons
- inappropriate applications of parts or accessories
- changes or interventions not previously authorized by the supplier
- usage of non-original spare parts
- non-observance of the instructions in this user's manual
- exceptional incidents

INTENDED USE

... the second cradle ... more comfortable than a cradle ...

What can the patient expect?

Cosiness: **culla-care** is a therapy- and relax chair. It offers the patient cosiness, relax and warmth.

Mobility: **culla-care** is easily manoeuvrable and with its blockable caster wheels extremely mobile. The sitting/lying position is easily regulated.

Safety: its nest-like shape adapts to the body, it gives sustain, safety and nest-like warmth.

Comfort: the perfect form and the high quality upholstery secure an optimal position from sitting to lying. The comfortable front access facilitates the active transfer.

Who needs culla-care?

culla-care is a relief for people with physical and mental disabilities as well as for elderly users. It helps people with muscular problems, with decubitus and it calms down patients suffering from Alzheimer and Parkinson.


culla-care improves the quality of life at home, in nursing homes, retiring homes and hospitals.

culla-care was designed and made to offer the patient cosiness and relax; the patient is back in his childhood and the cosy cradle shape of **culla-care** gives the user a sensation of being protected. He definitely calms down.

culla-care can only be used for the applications mentioned above. The manufacturer assumes no responsibility for other applications and for damages caused by improper use.

We wish you a good success with the use of **culla-care**.

COMPONENTS

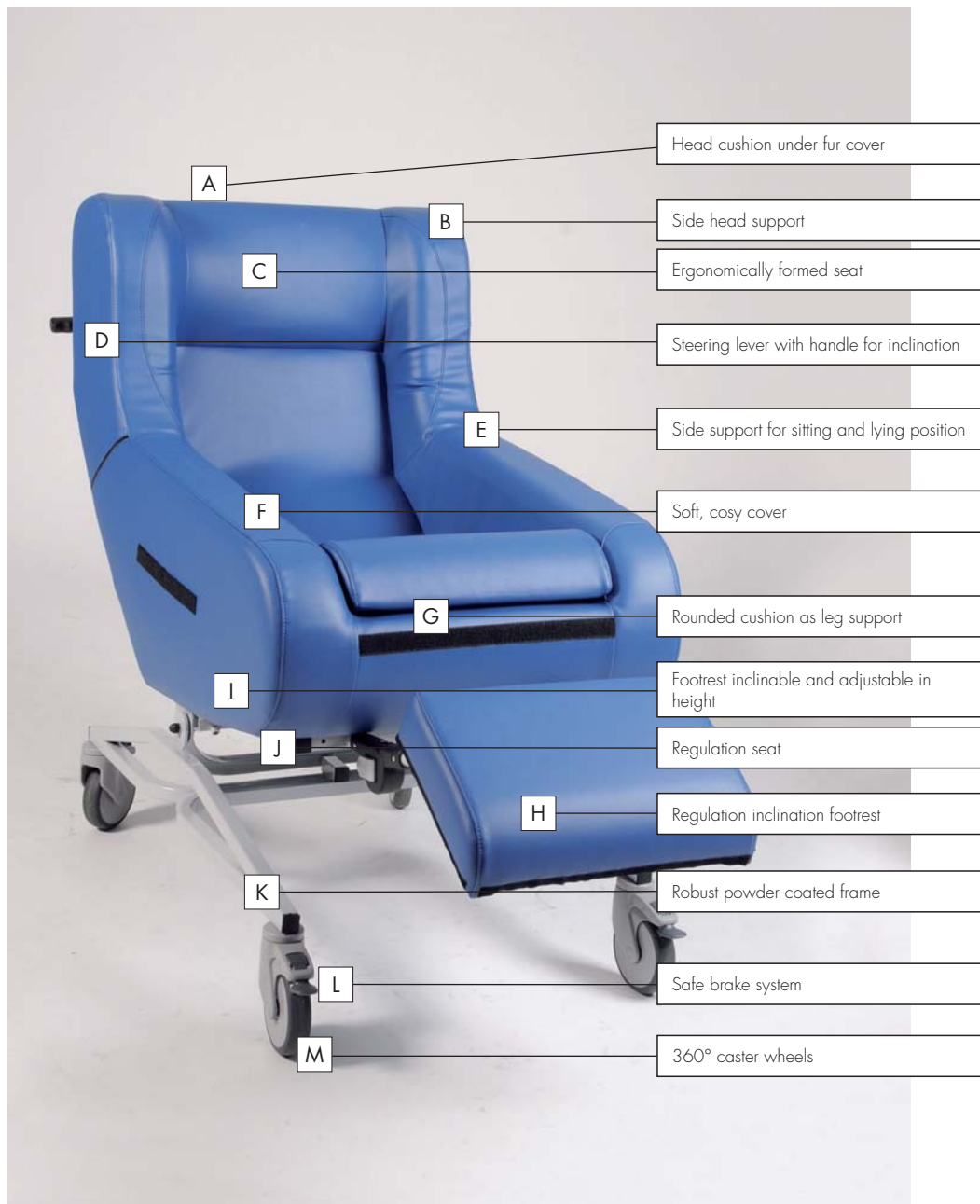ADJUSTMENT AND OPERATION

fig. 1

fig. 2

fig. 3

fig. 4

fig. 5

fig. 6

fig. 7

fig. 8

Press the inclination lever (pos. D page 25) and put **culla-care** in the upright position for a perfect access. fig. 2. Approach the patient to guarantee an easy access fig. 1.

Please pay attention that the footrest (pos. H page 25) is under the seat fig. 3. To position the footrest use the side lever (pos. J page 25) loosen the handle, after reaching the desired height tighten the handle again fig. 4.

When the patient is sitting comfortably fig. 5, you can press the inclination lever and put the patient in a position ranging from sitting to lying (pos.D page 25) fig. 2. Find the best position in height and inclination. Positions are adaptable to the needs of every single user. The adjustment of the back and the seat remain in an orthopaedic and anatomic correct position independently from inclination of the back fig. 6.

Put the footrest in the desired height and inclination by loosening the side handle(pos. J page 25) fig. 4. Tighten it again after reaching the desired position.

After all these preparations it is time to start with the introduction to the use of **culla-care**.
No pre-regulations made!

It is important to get the patient to know the aid and its functions. Normally patients are not afraid of using it; very often they react with enthusiasm and pleasure. Persons who tend to have vehement and uncontrollable movements should never be left unattended for safety reasons.

For „anxious“ people, who are afraid of using **culla-care**, the following procedure is advisable: when the patient is trying **culla-care** for the first time he should not be left alone, but should be accompanied by qualified personnel or the therapist and a family member. They place themselves in front, next to or behind the patient, they convince the patient to sit down carefully and then move **culla-care** slowly and with caution.

These cautionary measures give the patient a great sense of security, especially if at the same time he gets an explanation of what is happening.

Initially it is necessary to try to reach a movement by pushing at constant speed. Limited distances, whether for “daring” or for “anxious” patients, are advisable, as they produce a useful effect of orientation and progress.

ASSEMBLY

culla-care functions in a very easy and secure way: even though a good introduction by the dealer / therapist is necessary. There are instructions to be considered for the optimal use of this device.

Start-up:

culla-care is delivered in one part:

- the single pants are
- base with caster wheels and brakes
- upper part with nest-like seating
- head cushion
- reduction cushions if necessary
- footrest

The soft cover in terry towel, lamb's wool or artificial lamb fur can be put over the seating and easily fixed with velcro at the back on two sides and in front.

ACCESSORIES

Together with culla-care the following accessories are available:

CC5003 side-table inclinable with fixture fig. A

CC5004 infusion stand with fixture fig. B

CC5005 side support 1 piece fig. D

CC5006 cushion to reduce height and depth fig. C

fig. A

mount on the frame and screw up tightly

fig. B

mount on the frame and screw up tightly

fig. C

place the conical reduction cushion in the therapy chair to reduce height and depth

fig. D

place the side support left or/and right in the therapy chair to support the pelvis and to reduce superfluous space at the sides

TYPES

basic model culla-care

- culla-care** CC4000 therapy chair and day bed **culla-care**
 - colour upholstery green
- culla-care** CC4000 therapy chair and day bed **culla-care**
 - colour upholstery red
- culla-care** CC4000 therapy chair and day bed **culla-care**
 - colour upholstery yellow
- culla-care** CC4000 therapy chair and day bed **culla-care**
 - colour upholstery blue
- culla-care** CC4004 OUTDOOR therapy chair and day bed with big rear wheels and brake
 - colour upholstery choice (blue, red, yellow or green))

CLEANING INSTRUCTIONS ACCESSORIES

- CC5000 artificial lamb fur cover washable (follow washing instructions)
- CC5001 terry towel cover washable (follow washing instructions)
- CC5002 lamb's wool cover (follow washing instructions)
- CC5003 side-table inclinable with fixture
- CC5004 infusions stand with fixture
- CC5005 reduction cushion as side support 1 piece
- CC5006 reduction cushion to reduce height and depth

Please find more information in our catalogue!

It is strictly forbidden to make any tilt adjustments or change parts or accessories with the user in the therapy chair culla-care!

Cleaning instructions - seat: clean with damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches or synthetic detergents

Cleaning instructions - metal parts: keep all metal and upholstered parts clean and wipe them down with a soft and damp cloth and dry

Cleaning instructions - caster wheels: keep the caster wheels clean; remove dust, hairs and other substances carefully

TECHNICAL DATA

MAINTENANCE AND TECHNICAL DATA

Wheel dimensions:

wheel dimensions front: 125 x 30 mm indoor, 125 x 30 mm outdoor

wheel dimensions rear: 150 x 30 mm indoor, 400 x 40 mm outdoor

Max. user weight: 150 kg

Accessories:

For any substitutions of accessories please refer to the instructions. Remember that for dismounting follow the instructions in inverse order.

Maintenance:

Maintenance has to be performed by the dealer or its representative once a year.

SPARE PARTS

Never make any substitutions of parts or accessories while the patient is using **culla-care!**

Any product liability from the side of the manufacturer (Elmi Srl, Zona Artigianale 25, 39052 Caldaro, BZ, Italy) is excluded from the beginning in such cases.

For questions on spare parts and accessories please contact your authorized dealer or the manufacturer directly.

tel: 00390471961002 or via e-mail: info@elmi-reha.it.

Environmental factors:

All packing materials are recyclable, as carton and paper are used.

If plastic foils are eventually used they are made in PE and therefore also recyclable.

Elmi GmbH-Srl
Handwerkerzone 25 Zona Artigianale
39052 Kaltern-Caldaro (BZ) · Italy
Tel. +39 0471 961002 · info@elmi-reha.it